**D** **B E D I E N U N G S A N L E I T U N G**

**GB** **O P E R A T I O N I N S T R U C T I O N S**

**F** **M O D E D ' E M P L O I**

# MA 30

## Inhaltsverzeichnis / Index / Liste

### Vorwort / Preface / Préface

Seite 2

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### Seite 3 – 8

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### Operation Instructions **GB**

- Warning notices
- Scope of delivery + startup
- Maintenance
- Questions + explanation of symbols
- Declaration of conformity

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**GB**

## **Preface**

We are pleased that you have chosen a Muss Dental product. These instructions will enable you to become familiar with the unit and its qualities. You will also find suggestions which have been compiled to ensure long and problem-free operation. Please read these instructions carefully before using the unit.

The technical specifications, illustrations and dimensions contained in these instructions are not binding. No claims may be derived from them. We reserve the right to make technical improvements without amending these instructions.

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## Technical Data

Max. Normal Output:	150 watt
Rated Voltage:	230 volts / 0,8A/T 115 volts / 1,8A/T
Rated Frequency:	50 Hz / 60 Hz
Noise Level:	67 dB(A)
Revolutions:	1.000 - 30.000 rpm
Handpiece:	with carbon brushes
Chuck:	Ø 2,35 mm or Ø 3,00 mm

**Please read the instructions carefully before operating the unit, and keep it for later use.**

## Warning!

The instructions are to be carefully read and followed to ensure proper operation and handling of the unit. Failure to do so could result in serious injuries and/or damage to the unit.

- These operation instructions are part of the unit.
- Please keep it during the time the product lasts.
- Please pass these operation instructions on to any successive possessor/user.

If other languages of these operation instructions may be needed, please contact the manufacturer. Any copying or transmission of these operation instructions to a third party is prohibited without the specific approval of the manufacturer.

The manufacturer cannot be held responsible for any damages resulting from:

- External interference (bad quality of the media or poor installation)
- Improper operation of unit
- Improper repairs

Interference by an unauthorised party would nullify any implied warranty. The manufacturer recommends using only new authorised spare-parts for any repairs needed. As safe and proper operation of the unit can only be guaranteed by correct use of the relevant b, burs as outlined in the instructions.

## Scope

The manufacturer's guidelines for the instruments need to be followed concerning the highest speed allowance and the contact pressure (shaft of  $\varnothing$  2,35 or  $\varnothing$  3,0 mm). The highest speed allowance of 30.000 rpm may not be exceeded. Please only use rotating instruments from trade-marked manufacturers and take care of the technical data, only the correct tools/burs need to be used by working with a speed of 30.000 rpm.

## Accident Prevention Regulations are to be followed!

The product needs to be kept indoor of normal room temperature and normal humidity and not to be used in wet/humid rooms.

Primary function: for grinding purposes in the dental laboratories, only to be used by trained dental technicians.

Goggles are always necessary when working with rotating instruments.

We advise you to use a dust aspirator or use a mask by working with plaster

GB

# OPERATION INSTRUCTION

## Accident Prevention Regulations are to be followed!

If the handpiece is not in use, it should be placed on the handpiece holder, or on any other proper holder.

If the unit is not in proper condition or is not being handled correctly, it is possible that injuries may occur or cause damage to the material and unit.

## Unplug the Unit before Opening the Housing

The repair and maintenance of the electrical parts of the unit may only be done by professionally skilled and trained workers, informed of the safety guidelines. New spare parts, used in repairing the unit must be in accordance with spare-parts with the manufacturer required specifications.

## Scope of Delivery/Dimensions: F203 / H240B

Operating unit, power cord, handpiece, handpiece holder, chuck key, cleaning brush, operation instructions, certificate of warranty.

Measurement control unit: W 23 cm x D 26 cm x H 15 cm, weight: 3,1 kg  
Measurement handpiece: L 18 cm, ø 3 cm, weight: 280 g

## Scope of Delivery/Dimensions: K203 / H240B

Operating unit, power cord, handpiece, handpiece holder, chuck key, cleaning brush, drill template, hanging, 2 screws, operation instructions, certificate of warranty.

Measurement control unit: W 33 cm x D 23 cm x H 10 cm, weight: 3,5 kg  
Measurement handpiece: L 18 cm, ø 3 cm, weight: 280 g

## Scope of Delivery/Dimensions: T203 / H240B

Operating unit, power cord, handpiece, treadle switch, handpiece holder, chuck key, cleaning brush, operation instructions, certificate of warranty.

Measurement control unit: W 33 cm x D 23 cm x H 9 cm, weight: 3,5 kg  
Measurement handpiece: L 18 cm, ø 3 cm, weight: 280 g

## Startup (according to the scope of delivery)

- Remove the device completely out of the packaging.
- Set up the table top control unit at an adequate workstation.
- Set up the foot control unit at an adequate workstation.
- Mount the knee control unit onto your table/bench according to the separate instructions (written on the drill template MU 3061).

## Cable Connections

- Connect and block the handpiece at the back of the control unit at the hub .
- **Only for table top control units:** Connect and block the treadle switch at the back of the control unit at the hub .
- Insert the power cord into the back of the control unit. Plug the plug into the power socket.

## Operation of Handpiece: H240B

Change the burs only when the handpiece has come to a complete stop. No warranty by disregard! Turn the grip sleeve clockwise in order to exchange the desired burs to open the mechanism (for chuck). Turn the grip sleeve counter clockwise until it snaps in order to close it again.

## Operation of Foot Control Unit: F203

Activate the foot lever into the direction of the max. speed, so the rate of revolutions will be continuously variable. Release the foot lever, the handpiece will stop.

# OPERATION INSTRUCTIONS

## Operation of Knee Control Unit: K203

Turn the control unit “on” with the on/off switch. The “green” pilot lamp lights up. Select the desired max. speed with the speed-selector lever by turning it clockwise. The selected speed is being shown at the LED reading. The rate of revolutions will be continuously variable to the set speed. The handpiece comes to a stop by releasing the knee lever. By using the reversal of rotation switch , the direction of course of the handpiece will change (lamp lights up).

## Operation of Table Top Control Unit: T203

Turn the control unit “on” with the on/off switch. The “green” pilot lamp lights up. Select the desired max. speed with the speed-selector lever by turning it clockwise. By operating the treadle switch, the selected speed will be reached, which will be shown at the LED reading. In order to operate the unit without the treadle switch, the treadle switch just needs to be unplugged. The unit operates at the set max. speed. By using the reversal of rotation switch , the direction of course of the handpiece will change (lamp lights up).

**Do not clean handpiece with pressure air, also, the use of oil is prohibited!**

 Check the chuck at least once a month and clean it.

## Motor Overload Protection

In case of an overload, the built-in electronics will shut itself and the motor off. The unit will be ready for further use.

## Storage of the Product

The storage of the product needs to be guaranteed (normal room temperature and normal humidity). Keep the packaging for storage/transportation purposes.

**Electromagnetic compatibility of product is given, tested after EMV.**

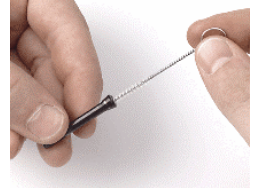
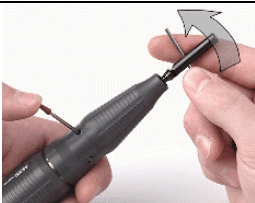
## MAINTENANCE OF THE CHUCK

1	<p><b>Removal of Small Cap:</b></p> <p>Remove the small cap MU 1444 out of the grip sleeve.</p>	
2	<p><b>Removal of the Bur:</b></p> <p>Turn the motor grip sleeve in direction of arrow in order to open the handpiece and remove the bur.</p> <p> <b>Never open handpiece while in use.</b></p>	
3	<p><b>Loosening of Chuck:</b></p> <p>Lock the motor shaft with any bur, and unscrew the chuck with chuck key MU 0222 (direction of arrow).</p>	
4	<p><b>Remove the Chuck:</b></p> <p>Remove the chuck out of the shaft, check it, clean it or replace it. Damaged and fooled chucks reduce the tension and the exact truth of rotation.</p> <p> <b>Risk of Accident!</b>          If you work with a defective chuck a risk of accident could be possible.</p>	

**GB**

# OPERATION INSTRUCTIONS

## MAINTENANCE OF THE CHUCK

5	<p><b>Cleaning of the Shaft:</b></p> <p>Remove the dirt with the cleaning brush MU 0492.</p>	
6	<p><b>Cleaning the Chuck:</b></p> <p>Clean the chuck MU 0221 (Ø 2,35 mm) with the cleaning brush MU 0492.</p> <p>Ø 3,00 mm chucks have the order number MU 0221 B.</p>	
7	<p><b>Insert the Chuck:</b></p> <p>Insert the chuck.</p>	
8	<p><b>Tighten Chuck:</b></p> <p>Block the motor shaft with any bur.</p> <p>Tighten the chuck with the chuck key MU 0222 (direction of arrow):</p>	
9	<p><b>Tightening of Bur:</b></p> <p>Insert the bur.</p> <p>Close the grip sleeve (direction of arrow).</p> <p> Just use unobjectionable burs!</p>	
10	<p><b>Insert Small Cap:</b></p> <p>Insert the small cap MU 1444 with your thumb.</p>	

 Please take a look at the warranty conditions which will be enclosed separately.

### Hints for Safe Disposal of Product

Instruction according "Waste Electrical and Electronic Equipment" WEEE-guide line 2002/96/EG. Please do not dispose the device in your household waste, but deliver it to a special place

GB

# OPERATION INSTRUCTIONS

## Questions and Answers

- **The speed can not be adjusted at the control unit:** Change of the electronics or the potentiometer (service engineer).
- **The chuck does not hold the burs anymore or bad whirling arm:** The chuck is dirty or broken (look at the maintenance instruction); check or replace it.
- **The handpiece makes strange noises:** The bur was not inserted correctly, or turn the handpiece sleeve to the end point or replace ball bearings (service engineer).

If other problems occur which are not listed under "Questions and Answers" please call the manufacturer:  
**+49 5103 70650**

## Manufacturer:

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**Am Weingarten 7**

**D-30974 Wennigsen**

**GERMANY**

**Tel.: +49 5103 70650**

**Fax: +49 5103 706579**

**e-mail: [muss.dental@t-online.de](mailto:muss.dental@t-online.de)**

**Internet: [www.muss-dental.de](http://www.muss-dental.de)**

## Explanations of the Symbols on the Product:

Motor hub

Hub for treadle switch

Clockwise/counter-clockwise rotation

## Explanations of the Symbols Mentioned in the Operation Instructions:

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Declare, that the product

MA 30 Micromotor-unit consisting of:  
MA 30 handpiece and cable type H240B

In connection with

MA 30 control unit type F203, K203  
Type T203 with treadle switch item- no. 9746

To which this declaration relates, is in conformity with the protection requirements according to the provisions of Directive

89/392/EWG  
73/23/EWG

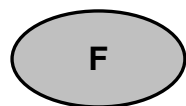
The following standards or other normative documents are applied for the judgement of this product:

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Nous MUSS DENTAL GmbH