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GB **OPERATION INSTRUCTIONS**

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MA 31

Inhaltsverzeichnis / Index / Liste

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GB

Preface

We are pleased that you have chosen a Muss Dental product. These instructions will enable you to become familiar with the unit and its qualities. You will also find suggestions which have been compiled to ensure long and problem-free operation. Please read these instructions carefully before using the unit.

The technical specifications, illustrations and dimensions contained in these instructions are not binding. No claims may be derived from them. We reserve the right to make technical improvements without amending these instructions.

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Order No.: 9527

Technical Data

Max. Normal Output:	180 watt
Rated Voltage:	230 volts / 2,5A/T 115 volts / 4A/T
Rated Frequency:	50 / 60 Hz
Noise Level:	53 dB(A)
Revolutions:	1.000 - 30.000 rpm
Handpiece:	brushless
Chuck:	Ø 2,35 mm or Ø 3,00 mm

Please read the instructions carefully before operating the unit, and keep it for later use.

Warning!

The instructions are to be carefully read and followed to ensure proper operation and handling of the unit. Failure to do so could result in serious injuries and/or damage to the unit.

- These operation instructions are part of the unit.
- Please keep it during the time the product lasts.
- Please pass these operation instructions on to any successive possessor/user.

If other languages of these operation instructions may be needed, please contact the manufacturer. Any copying or transmission of these operation instructions to a third party is prohibited without the specific approval of the manufacturer.

The manufacturer cannot be held responsible for any damages resulting from:

- External interference (bad quality of the media or poor installation)
- Improper operation of unit
- Improper repairs

Interference by an unauthorised party would nullify any implied warranty. Muss Dental recommends using only new authorised spare-parts for any repairs needed. As safe and proper operation of the unit can only be guaranteed by correct use of the relevant burs, as outlined in the instructions.

Scope

The manufacturer's guidelines for the instruments need to be followed concerning the highest speed allowance and the contact pressure (shaft of Ø 2,35 or Ø 3,0 mm). The highest speed allowance of 30.000 rpm may not be exceeded. Please only use rotating instruments from trade-marked manufacturers and take care of the technical data, only the correct tools/burs need to be used by working with a speed of 30.000 rpm.

Accident Prevention Regulations are to be followed!

The product needs to be kept indoor of normal room temperature and normal humidity and not to be used in wet/humid rooms.

Primary function: for grinding purposes in the dental laboratories, only to be used by trained dental technicians.

Goggles are always necessary when working with rotating instruments.

We advise you to use a dust aspirator or use a mask by working with plaster

GB

OPERATION INSTRUCTION

Accident Prevention Regulations are to be followed!

If the handpiece is not in use, it should be placed on the handpiece holder, or on any other proper holder.

If the unit is not in proper condition or is not being handled correctly, it is possible that injuries may occur or cause damage to the material and unit.

Unplug the unit before opening the housing

The repair and maintenance of the electrical parts of the unit may only be done by professionally skilled and trained workers, informed of the safety guidelines. New spare parts, used in repairing the unit must be in accordance with spare-parts with the manufacturer required specifications.

Scope of Delivery/Dimensions: F300 / H310

Operating unit, power cord, handpiece, handpiece holder, chuck key, cleaning brush, instructions, and certificate of warranty.

Measurement control unit: W 23 cm x D 26 cm x H 15 cm, weight: 3,1 kg
Measurement handpiece: L 17 cm, ø 3 cm, weight: 240 g

Startup

- Remove the device completely out of the packaging.
- Set up the foot control unit at an adequate workstation.

Cable Connections

- The handpiece needs to be connected and blocked at the back of the control unit at the hub .
- Insert the power cord into the back of the control unit. Plug the plug into the power socket.

Operation of Handpiece: H310

Change the burs only when the handpiece has come to a complete stop. No warranty by disregard! To open the mechanism (for chuck) you need to turn the grip sleeve clockwise in order to exchange the desired burs. In order to close it again, you need to turn the grip sleeve counter clockwise until it snaps.

Operation of Foot Control Unit: F300

The rate of revolutions will be continuously variable by activating the foot lever into the direction of max. speed. The handpiece stops by releasing the foot lever.

Do not clean handpiece with pressure air, also the use of oil is prohibited!

-  Check the chuck at least once a month and clean it.

Motor overload protection

In case of an overload, the built-in electronics will shut itself and the motor off. The unit will be ready for further use.

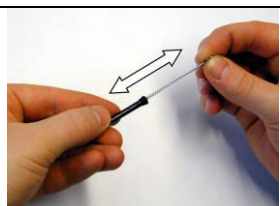
Storage of the product

The storage of the product needs to be guaranteed (normal room temperature and normal humidity). You need to keep the packaging for storage/transportation purposes.

Electromagnetic compatibility of product is given, tested after EMV.

GB OPERATION INSTRUCTIONS

MAINTENANCE OF THE CHUCK

1	<p>Removal of Motor End Cap:</p> <p>Unscrew motor end cap MU 4213 in direction of arrow. Carefully pull of plug.</p> <p>The removal of the cable is not necessary .</p>	
2	<p>Removal of Stopper:</p> <p>Remove stopper MU 1444 out of the grip sleeve.</p>	
3	<p>Removal of the Bur:</p> <p>To open the handpiece turn the motor grip sleeve in direction of arrow and remove bur.</p> <p> Never open handpiece while in use.</p>	
4	<p>Loosening of Chuck:</p> <p>Lock motor shaft with any bur, and stick it into the hole where you have removed the stopper MU 1444 and unscrew the chuck with chuck key MU 0222 (direction of arrow).</p>	
5	<p>Remove the Chuck:</p> <p>Remove chuck out of the shaft, check it, clean it or replace it. Damaged and fooled chucks reduce the tension and the exact truth of rotation.</p> <p> Risk of Accident! If you work with a defective chuck a risk of accident could be possible.</p>	
6	<p>Cleaning of the Shaft:</p> <p>Remove dirt with the cleaning brush MU 0492.</p>	
7	<p>Cleaning the Chuck:</p> <p>Clean the chuck MU 0221 (Ø 2,35 mm) with clean brush MU 0492. Ø 3,00 mm chucks have the order number MU 0221 B.</p>	

GB

OPERATION INSTRUCTIONS

MAINTENANCE OF THE CHUCK

8	Insert the Chuck: Insert chuck. Block motor shaft with any bur.	
9	Tighten Chuck: Tighten chuck with chuck key MU 0222 (direction of arrow):	
10	Tightening of Bur: Insert bur. Close grip sleeve (direction of arrow).  Just use unobjectionable burs!	
11	Mounting of Motor End Cap: Carefully place pins into the contacts. Screw in motor end cap MU 4213 (direction of arrow) and tighten it by hand.	
12	Insert Stopper: Push in stopper MU 1444 with your thumb.	

 Please take a look at the warranty conditions which will be enclosed separately.

Hints for safe disposal of product

Instruction according "Waste Electrical and Electronic Equipment" WEEE-guide line 2002/96/EG. Please do not dispose the device in your household waste, **but deliver it to a special place**

Questions and Answers

- **The speed can not be adjusted at the control unit:** Change or adjust the electronics or the potentiometer (service engineer).
- **The chuck does not hold the burs anymore or bad whirling arm:** The chuck is dirty or broken (look at the maintenance instruction); check or replace it.
- **The handpiece makes strange noises:** The bur was not inserted correctly, or turn the handpiece sleeve to the end point, or replace ball bearings (service engineer).

If other problems occur which are not listed under "Questions and Answers" please call the manufacturer:
+49 5103 70650

GB

OPERATION INSTRUCTIONS

Manufacturer:

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Am Weingarten 7

D-30974 Wennigsen

GERMANY

Tel.: +49 5103 70650

Fax: +49 5103 706579

e-mail: muss.dental@t-online.de

Internet: www.muss-dental.de

e-mail: muss.dental@t-online.de Internet: www.muss-dental.de

Explanations of the Symbols on the Product:

 Motor hub

Explanations of the Symbols Mentioned in the Operation Instructions:

-  Generic warning
-  Cutting hazard warning
-  Arc flash (danger electricity)
-  Eye protection must be worn
-  General mandatory sign
-  Information

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Declare, that the product

MA 31 Micromotor-unit consisting of:
MA 31 handpiece and cable type H310

In connection with

MA 31 control unit type F300

To which this declaration relates, is in conformity with the protection requirements according to the provisions of Directive

89/392/EWG
73/23/EWG

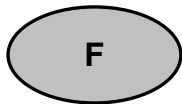
The following standards or other normative documents are applied for the judgement of this product:

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Nous MUSS DENTAL GmbH