D

B E D I E N U N G S A N L E I T U N G

GB

O P E R A T I O N I N S T R U C T I O N S

F

M O D E D ' E M P L O I

MA 50

Inhaltsverzeichnis / Index / Liste

Vorwort / Preface / Préface

Seite 2

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Seite 3 – 8

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Operation Instructions GB

page 8 – 12

- warning notices
- scope of delivery + start up
- maintenance
- questions + explanation of symbols
- declaration of conformity

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GB

Preface

We are pleased that you have chosen a Muss Dental product. These instructions will enable you to become familiar with the unit and its qualities. You will also find suggestions which have been compiled to ensure long and problem-free operation. Please read these instructions carefully before using the unit.

The technical specifications, illustrations and dimensions contained in these instructions are not binding. No claims may be derived from them. We reserve the right to make technical improvements without amending these instructions.

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Technical Data

Max. Normal Output:	260 watt
Rated Voltage:	230 volts / 2,5A/T 115 volts / 4A/T
Rated Frequency:	50 Hz / 60 Hz
Noise Level:	64 dB(A)
Revolutions:	1.000 - 50.000 rpm
Handpiece:	brushless
Chuck:	Ø 2,35 mm or Ø 3,00 mm

Please read the instructions carefully before operating the unit, and keep it for later use.

Warning!

The instructions are to be carefully read and followed to ensure proper operation and handling of the unit. Failure to do so could result in serious injuries and/or damage to the unit.

- These operation instructions are part of the unit.
- Please keep it during the time the product lasts.
- Please pass these operation instructions on to any successive possessor/user.

If other languages of these operation instructions may be needed, please contact the manufacturer. Any copying or transmission of these operation instructions to a third party is prohibited without the specific approval of the manufacturer.

The manufacturer cannot be held responsible for any damages resulting from:

- External interference (bad quality of the media or poor installation)
- Improper operation of unit
- Improper repairs

Interference by an unauthorised party would nullify any implied warranty. Muss Dental recommends using only new authorised spare-parts for any repairs needed. As safe and proper operation of the unit can only be guaranteed by correct use of the relevant burs, as outlined in the instructions.

Scope

The manufacturer's guidelines for the instruments need to be followed concerning the highest speed allowance and the contact pressure (shaft of Ø 2,35 or Ø 3,0 mm). The highest speed allowance of 50.000 rpm may not be exceeded. Please only use rotating instruments from trade-marked manufacturers and take care of the technical data, only the correct tools/burs need to be used by working with a speed of 50.000 rpm.

Accident Prevention Regulations are to be followed!

The product needs to be kept indoor of normal room temperature and normal humidity and not to be used in wet/humid rooms.

Primary function: for grinding purposes in the dental laboratories, only to be used by trained dental technicians.

Goggles are always necessary when working with rotating instruments.

We advise you to use a dust aspirator or use a mask by working with plaster

GB

OPERATION INSTRUCTION

Accident Prevention Regulations are to be followed!

If the handpiece is not in use, it should be placed on the handpiece holder, or on any other proper holder.

If the unit is not in proper condition or is not being handled correctly, it is possible that injuries may occur or cause damage to the material and unit.

Unplug the unit before opening the housing

The repair and maintenance of the electrical parts of the unit may only be done by professionally skilled and trained workers, informed of the safety guidelines. New spare parts, used in repairing the unit must be in accordance with spare-parts with the manufacturer required specifications.

Scope of Delivery/Dimensions: F501 / H550

Operating unit, power cord, handpiece, handpiece holder, chuck key, box spanner, cleaning brush, operation instructions, certificate of warranty.

Measurement control unit: W 23 cm x D 26 cm x H 15 cm, weight: 3,1 kg

Measurement handpiece: L 17 cm, ø 3 cm, weight: 280 g

Scope of Delivery/Dimensions: K501 / H550

Operating unit, power cord, handpiece, handpiece holder, chuck key, box spanner, cleaning brush, drill template, hanging, 2 screws, operation instructions, certificate of warranty.

Measurement control unit: W 33 cm x D 23 cm x H 10 cm, weight: 3,5 kg

Measurement handpiece: L 17 cm, ø 3 cm, weight: 280 g

Scope of Delivery/Dimensions: T501 / H550

Operating unit, power cord, handpiece, treadle switch, handpiece holder, chuck key, box spanner, cleaning brush, operation instructions, certificate of warranty.

Measurement control unit: W 33 cm x D 23 cm x H 9 cm, weight: 3,5 kg

Measurement handpiece: L 17 cm, ø 3 cm, weight: 280 g

Startup (according to the scope of delivery)

- Remove the device completely out of the packaging.
- Set up the table top control unit at an adequate workstation.
- Set up the foot control unit at an adequate workstation.
- Mount the knee control unit onto your table/bench according to the separate instructions (written on the drill template MU 3061).

Cable Connections

- The handpiece needs to be connected and blocked at the back of the control unit at the hub .
- **Only for table top control units:** The treadle switch needs to be connected and blocked at the back of the control unit at the hub .
- Insert the power cord into the back of the control unit. Plug the plug into the power socket.

Operation of Handpiece: H550

Change the burs only when the handpiece has come to a complete stop. No warranty by disregard! To open the mechanism (for chuck) you need to turn the grip sleeve clockwise in order to exchange the desired burs. In order to close it again, you need to turn the grip sleeve counter clockwise until it snaps.

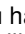
Operation of Foot Control Unit: F501

The rate of revolutions will be continuously variable by activating the foot lever into the direction of max. speed. The handpiece stops by releasing the foot lever. With the black button one can choose the max. speed of 30.000 rpm or 50.000 rpm, the lamp lights up red if you have chosen 50.000 rpm.

OPERATION INSTRUCTIONS

Operation of Knee Control Unit: K501

The control unit needs to be turned "on" with the on/off switch. The "green" pilot lamp lights up. Select the desired max. speed with the speed-selector lever by turning it clockwise. The selected speed is being shown at the LED reading. The rate of revolutions will be continuously variable to the set speed. The handpiece comes to a stop by releasing the knee lever. With the 50.000 rpm button one can choose the max. speed of 30.000 rpm or 50.000 rpm, the lamp lights up yellow if you have chosen 50.000 rpm. By using the reversal of rotation switch , the direction of course of the handpiece will change (lamp lights up). The max. speed will be 30.000 rpm.

Operation of Table Top Control Unit: T501

The control unit needs to be turned "on" with the on/off switch. The "green" pilot lamp lights up. Select the desired max. speed with the speed-selector lever by turning it clockwise. By operating the treadle switch, the selected speed will be reached, which will be shown at the LED reading. In order to operate the unit without the treadle switch, the treadle switch just needs to be unplugged. The unit operates at the set max. speed. With the 50.000 rpm button one can choose the max. speed of 30.000 rpm or 50.000 rpm, the lamp lights up yellow if you have chosen 50.000 rpm. By using the reversal of rotation switch , the direction of course of the handpiece will change (lamp lights up). The max. speed will be 30.000 rpm.

Do not clean handpiece with pressure air, also, the use of oil is prohibited!

 Check the chuck at least once a month and clean it.

Motor overload protection

In case of an overload, the built-in electronics will shut itself and the motor off. The unit will be ready for further use.

Storage of the product

The storage of the product needs to be guaranteed (normal room temperature and normal humidity). You need to keep the packaging for storage/transportation purposes.

Electromagnetic compatibility of product is given, tested after EMV.

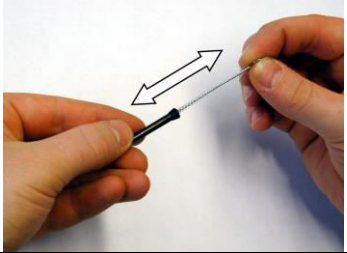
MAINTENANCE OF THE CHUCK

<p>1</p>	<p>Removal of the Bur:</p> <p>To open the handpiece turn the motor grip sleeve in direction of arrow and remove bur.</p> <p> Never open handpiece while in use.</p>	
<p>2</p>	<p>Removal of Motor End Cap:</p> <p>Unscrew motor end cap MU 4115 in direction of arrow.</p>	
<p>3</p>	<p>Loosening of the Chuck:</p> <p>Unscrew cap MU 4199 from the handpiece. Hold shaft of the motor with box spanner MU 4136 and unscrew the chuck with the chuck key MU 0222 (direction of arrow).</p>	

GB

OPERATION INSTRUCTIONS

MAINTENANCE OF THE CHUCK

4	<p>Remove the Chuck:</p> <p>Remove chuck out of the shaft, check it, clean it or replace it. Damaged and fooled chucks reduce the tension and the exact truth of rotation.</p> <p> Risk of Accident! If you work with a defective chuck a risk of accident could be possible.</p>	
5	<p>Cleaning of the Shaft:</p> <p>Remove dirt with the cleaning brush MU 0492.</p>	
6	<p>Cleaning the Chuck:</p> <p>Clean the chuck MU 0221 (Ø 2,35 mm) with clean brush MU 0492.</p> <p>Ø 3,00 mm chucks have the order number MU 0221 B.</p>	
7	<p>Insert the Chuck:</p> <p>Lock shaft of the motor with the box spanner MU 4136 and screw in the chuck with the chuck key MU 0222 (direction of arrow).</p> <p>Screw on cap MU 4198.</p>	
8	<p>Mounting of Motor End Cap:</p> <p>Carefully place pins into the contacts. Screw in motor end cap MU 4115 and tighten it by hand (direction of arrow).</p>	
9	<p>Tightening of the Burs:</p> <p>Insert the bur and turn handpiece housing (direction of arrow). Only use unobjectionable burs.</p>	

 Please take a look at the warranty conditions which will be enclosed separately.

GB

OPERATION INSTRUCTIONS

Hints for safe disposal of product

Instruction according "Waste Electrical and Electronic Equipment" WEEE-guide line 2002/96/EG. Please do not dispose the device in your household waste, **but deliver it to a special place**

Questions and answers

- **The speed can not be adjusted at the control unit:** Change or adjust the electronics or the potentiometer (service engineer).
- **The chuck does not hold the burs anymore or bad whirling arm:** The chuck is dirty or broken (look at the maintenance instruction); check or replace it.
- **The handpiece makes strange noises:** The bur was not inserted correctly, or turn the handpiece sleeve to the end point, or replace ball bearings (service engineer).
- **Error shown in display:**

E1	Overcurrent
	- Motor shorted
	- Motor cable broken
	-
	- Motor cable not connected
	- Motor cable in control unit broken
E2	Start-up commutation error
	- Overload protection was activated
	- Start-up failure
E3	Motor uses more than 42,5 V
E4	Low voltage in mains
E5	Temperature above 70°C (data is measured on the circuit board).

If other problems occur which are not listed under "Questions and Answers" please call our hotline: +49 5103 70650

Manufacturer:

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Am Weingarten 7

D-30974 Wennigsen

GERMANY

Tel.: +49 5103 70650

Fax: +49 5103 706579

e-mail: muss.dental@t-online.de

Internet: www.muss-dental.de

Explanations of the Symbols on the Product:

-  Motor hub
-  Hub for treadle switch
-  Clockwise/counter-clockwise rotation
-  more than 30.000 rpm

Explanations of the Symbols Mentioned in the Operation Instructions:

-  Generic warning
-  Cutting hazard warning
-  Arc flash (danger electricity)
-  Eye protection must be worn
-  General mandatory sign
-  Information

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CE

Declare, that the product

MA 50 Micromotor-unit consisting of:
MA 50 handpiece and cable type H550

In connection with

MA 50 control unit type F501, K501
Type T501 with treadle switch item-no. 9746

To which this declaration relates, is in conformity with the protection requirements according to the provisions of Directive

89/392/EWG
73/23/EWG

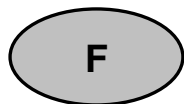
The following standards or other normative documents are applied for the judgement of this product:

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Nous MUSS DENTAL GmbH